

## How to Charge Your Battery Free Signia Hearing Aid



**Step 1:** You will know your hearing aid needs to be charged when you hear a low battery signal that repeats every 15 minutes.



You will hear an alert signal when the battery is low.

The signal is repeated every 15 minutes.

**Step 2:** Connect the charger to the power supply. The charger is ready when you see two green LED lights.



**Step 3:** Put the hearing aids in the charging slots



**Step 4:** The status indicator lights tell you how much charge remains. One orange bar = 33%. Two orange bars = 66%. Three orange bars = 99%.



**Step 5:** Your hearing aids are fully charge when all three bars are green.



**Step 6:** Take the hearing aids out of the charger and they turn on automatically.



**Step 7:** The LED lights on the charger convey additional information. Blinking orange = cooling phase. Permanently or blinking red: = error message. Please refer to your user guide. No LED light = the charger is not connected to a power supply or the hearing aids are not inserted correctly.

