

# RIC Receiver Instruction Guide



## How to replace a receiver

Locate the red receiver for the RIGHT hearing aid OR the blue receiver for LEFT hearing aid.

### Remove the receiver

- 1 Hold the hearing aid between your thumb and forefinger. (0:01)
- 2 Grasp the receiver near the top of the hearing aid and gently pull out. (0:02)



(0:01)



(0:02)

### Attach the receiver

- 1 Hold the hearing aid between your thumb and forefinger. (0:03)
- 2 Holding the new receiver near the plug, gently insert it into the hearing aid until it clicks into place. (0:06)
- 3 Give the receiver a gentle pull to ensure that it is properly secure.



(0:03)



(0:06)

To see video of this process, [please click here](#)

## How to insert a receiver

- 1 Position the hearing aid over the top of your ear, carefully placing it behind your ear and wrapping the tubing around the top of your ear. (0:02)
- 2 Use your forefinger to navigate the receiver bud into your ear canal. (0:05)
- 3 Gently push the bud into your ear canal. (0:06)



(0:02)



(0:05)



(0:06)

To see video of this process, [please click here](#)